

Computation Constants

Thermodilution (Bath) Cardiac Output

SWAN-GANZ CATHETER MODEL	INJECTATE TEMPERATURE								
	COLD INJECTATE (0-5°C)			ROOM INJECTATE (19-22°C)			ROOM INJECTATE (23-25°C)		
	10 cc	5 cc	3 cc	10 cc	5 cc	3 cc	10 cc	5 cc	3 cc
096F6	0.547	0.259	0.144	0.582	0.280	0.161	0.608	0.305	0.180
131F7	0.542	0.247	0.132	0.578	0.274	0.154	0.595	0.287	0.165
132F5	–	0.270	0.154	–	0.292	0.170	–	0.307	0.181
141HF7	0.542	0.247	0.132	0.578	0.274	0.154	0.595	0.287	0.165
143HTF7	0.554	0.259	–	0.587	0.286	–	0.599	0.291	–
C144F7, S144F7	0.547	0.252	0.134	0.579	0.277	0.156	0.597	0.295	0.169
C145HF6	0.547	0.252	0.134	0.579	0.277	0.156	0.597	0.295	0.169
151F7	0.542	0.247	0.132	0.578	0.274	0.154	0.595	0.287	0.165
139F75, 177F75, 744F75, 774F75	0.564	0.257	0.143	0.582	0.277	0.156	0.594	0.283	–
746F8, 777F8	0.550	0.256	–	0.585	0.282	–	0.600	0.292	–
831HF75	0.564	0.262	0.139	0.588	0.283	0.158	0.612	0.301	0.177
834HF75	0.564	0.257	0.143	0.582	0.277	0.156	0.607	0.294	0.170
931HF75	0.564	0.262	0.139	0.588	0.283	0.158	0.612	0.301	0.177
991HF8	0.568	0.268	0.147	–	–	–	0.616	0.302	0.176

CO-Set+ Closed Injectate Delivery System

SWAN-GANZ CATHETER MODEL	INJECTATE TEMPERATURE			
	COLD INJECTATE		ROOM INJECTATE	
	10 cc (6-12°C)	5 cc (8-16°C)	10 cc (18-25°C)	5 cc (18-25°C)
096F6	0.558	0.277	0.607	0.301
131F7	0.561	0.259	0.608	0.301
132F5	–	0.285	–	0.307
141HF7	0.561	0.259	0.608	0.301
143HTF7	0.569	0.266	0.589	0.287
C144F7	0.570	0.271	0.585	0.287
C145HF6, S145HF6	0.570	0.271	0.585	0.287
151F7	0.561	0.259	0.608	0.301
139F75, 177F75, 744F75, 774F75	0.574	0.287	0.595	0.298
746F8, 777F8	0.559	0.263	0.602	0.295
831HF75	0.578	0.272	0.592	0.290
834HF75	0.574	0.287	0.595	0.298
931HF75	0.578	0.272	0.592	0.290
991HF8	0.553	0.277 (8-12°C)	0.607	0.295



Edwards

For professional use. CAUTION: Federal (United States) law restricts this device to sale by or on the order of a physician. See instructions for use for full prescribing information, including indications, contraindications, warnings, precautions, and adverse events.

Edwards Lifesciences devices placed on the European market meeting the essential requirements referred to in Article 3 of the Medical Device Directive 93/42/EEC bear the CE marking of conformity.

Edwards, Edwards Lifesciences, the stylized E logo, CO-Set+, and Swan-Ganz are trademarks of Edwards Lifesciences Corporation.

© 2013 Edwards Lifesciences Corporation.
All rights reserved. AR09947

Edwards Lifesciences | edwards.com
Irvine, California 92614 USA

