

Acumen Analytics software sample report

Cohort Comparison

Acumen Analytics

Before HPI

Hypotension statistics

Gender	46.67 % (male), 53.33 % (female)
Age	58.67 ± 22.35
Weight	72.58 ± 24.97 kg, 160.07 ± 55.16 lbs
Height	163.07 ± 10.37 cm, 64.27 ± 4.15 in
Total monitoring time of the cohort	4008.67 minute(s)
Monitoring time per patient	267.24 ± 178.88 [185, 240, 349.83] minute(s)
Number of patients with hypotension	15 of 15 100%
Total number of hypotensive events in dataset	75 event(s)
Average number of hypotensive events per patient	5 ± 6.18 [2, 2, 6.5] event(s)
Average duration of each hypotensive event	9.48 ± 20.98 [1.67, 4, 6.33] minute(s)
Total duration of hypotension in cohort	711.32 minute(s)
Mean MAP under 65 mmHg per patient	58.89 ± 2.88 [56.43, 59.23, 61.24] mmHg
Area under 65 mmHg for MAP per patient (AUT)	331.4 ± 646.18 [43.42, 77, 267.67] mmHg x minute(s)
TWA* (MAP < 65 mmHg) per patient	1.39 ± 1.78 [0.15, 0.91, 2.1] mmHg
% of patients that experience an event under 50 mmHg	6 of 15 40%
Total number of events when a patient is under 50 mmHg	11 event(s)
Lowest MAP among all patients	36 mmHg

* TWA is time weighted average.

After HPI

Hypotension statistics

Gender	45.45 % (male), 54.55 % (female)
Age	54.82 ± 9.48
Weight	99.38 ± 37.73 kg, 219.18 ± 83.27 lbs
Height	168.09 ± 9.57 cm, 66.18 ± 3.68 in
Total monitoring time of the cohort	2792.67 minute(s)
Monitoring time per patient	253.88 ± 93.87 [182.58, 230, 270.58] minute(s)
Number of patients with hypotension	5 of 11 45.45%
Total number of hypotensive events in dataset	10 event(s)
Average number of hypotensive events per patient	0.91 ± 1.3 [0, 0, 1.75] event(s)
Average duration of each hypotensive event	2.1 ± 1.26 [1, 1.67, 3] minute(s)
Total duration of hypotension in cohort	20.99 minute(s)
Mean MAP under 65 mmHg per patient	61.16 ± 1.73 [59.93, 61.3, 62.46] mmHg
Area under 65 mmHg for MAP per patient (AUT)	8.85 ± 15.88 [0, 0, 10.42] mmHg x minute(s)
TWA* (MAP < 65 mmHg) per patient	0.04 ± 0.06 [0, 0, 0.06] mmHg
% of patients that experience an event under 50 mmHg	0 of 11 0%
Total number of events when a patient is under 50 mmHg	0 event(s)
Lowest MAP among all patients	55 mmHg



Edwards

Cohort Comparison


Acumen Analytics

Before HPI


Description

Cohort name	Before HPI
Number of files	15
First case start timestamp	April 15, 2019, 7:56:42AM
Last case start timestamp	May 01, 2019, 8:50:17AM


Total number of hypotensive events in cohort

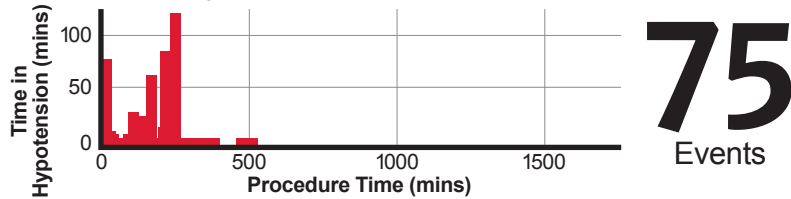

Percent of patients with hypotension

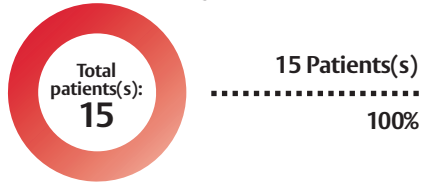

Average time in hypotension per procedure

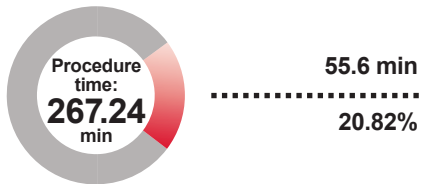


After HPI


Description

Cohort name	After HPI
Number of files	11
First case start timestamp	June 13, 2019, 10:07:29AM
Last case start timestamp	July 01, 2019, 11:23:41AM


Total number of hypotensive events in cohort

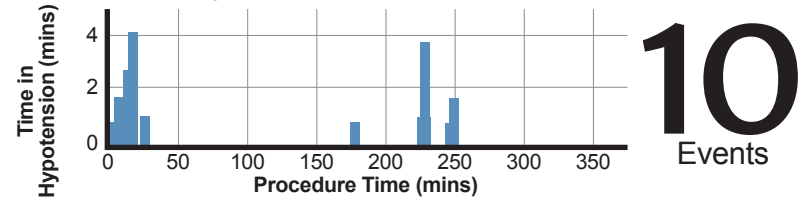

Percent of patients with hypotension

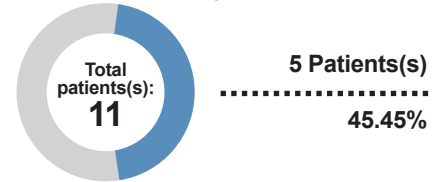

Average time in hypotension per procedure

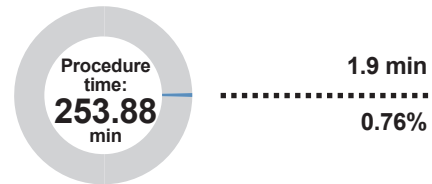

For professional use. For a listing of indications, contraindications, precautions, warnings, and potential adverse events, please refer to the Instructions for Use (consult eifu.edwards.com where applicable).

Edwards devices placed on the European market meeting the essential requirements referred to in Article 3 of the Medical Device Directive 93/42/EEC bear the CE marking of conformity. "Acumen Analytics is an educational tool, not a medical device."

Edwards, Edwards Lifesciences, the stylized E logo, Acumen, Acumen Analytics, HPI, and Hypotension Prediction Index are trademarks of Edwards Lifesciences Corporation or its affiliates. All other trademarks are the property of their respective owners.

© 2020 Edwards Lifesciences Corporation. All rights reserved. PP--EU-1236 v1.0

Edwards Lifesciences • Route de L'Etraz 70, 1260 Nyon, Switzerland • edwards.com



Edwards